



# UPDATING THE SICAT IMPLANT DATABASE




In order to perform the SICAT implant database update, you need to know whether you are using **Galileos Implant** or **SICAT Implant**. You can recognize **Galileos Implant** by starting the software out of SIDEKIS XG or SIDEKIS 4 using one of the icons on the left-hand side. You can recognize **SICAT Implant** by starting the software using the icon on the right hand side.

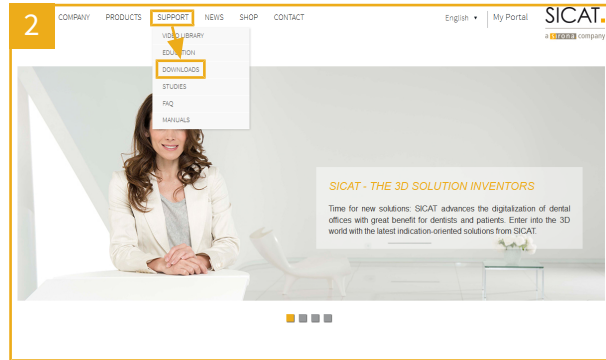


Open the SICAT homepage <http://www.sicat.com>. Select the **SUPPORT** area in the navigation bar and click **DOWNLOADS**.

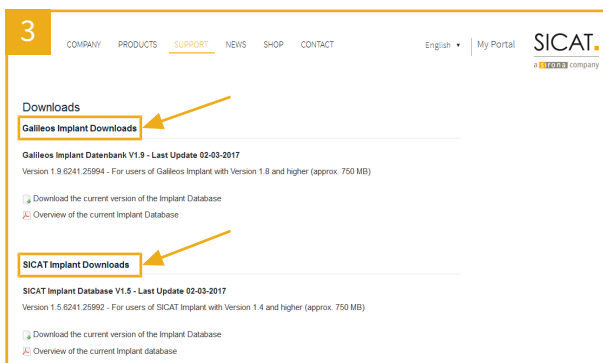


Scroll down until you have reached the **Galileos Implant Downloads** area or the **SICAT Implant Downloads** area that corresponds to your software. Click the **Download the current version of the Implant Database** link below the heading of the area that corresponds to your software.

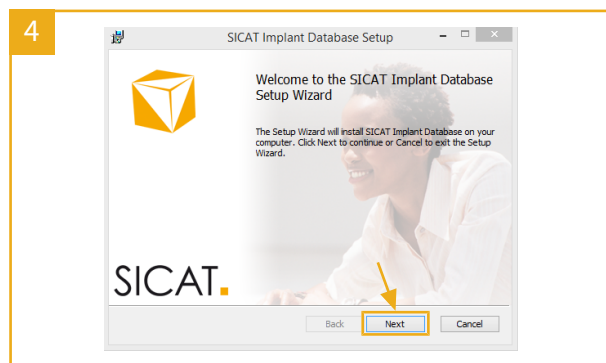

Open the downloaded MSI file. As soon as the welcome window opens, click the **Next** button.

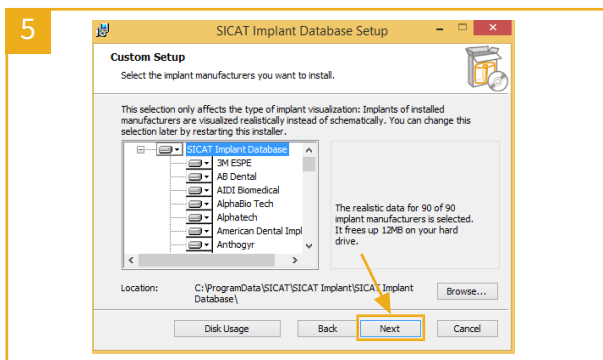

Please read and accept the license agreement by check marking the indicated box, then click **Next**. Afterwards, the installer shows a list of implant manufacturers. Usually, you do not have to make any changes to the list of implant manufacturers. Click the **Next** button. To start the actual installation, click the **Install** button in the next window.

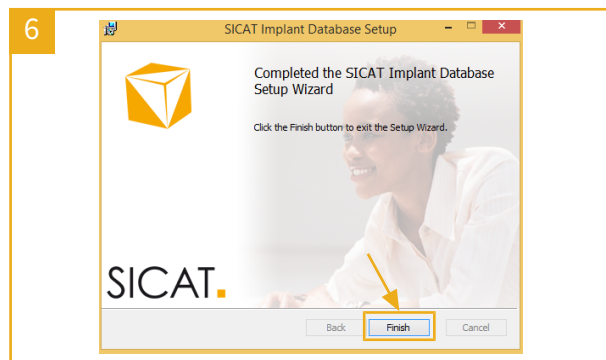

The installer removes old files which may exist and installs new files. Wait until the installer closes the window containing the progress bar and click the **Finish** button in the last window.

**SICAT.**

a **sirona** company

**MANUFACTURER**

**SICAT GMBH & CO. KG**

BRUNNENALLEE 6

53177 BONN, GERMANY

WWW.SICAT.COM

**LOCAL SUPPORT**

WWW.SICAT.COM/  
SUITESUPPORT

TEL: +49 6251 161670



DD2XQGD002

REVISION: 2017-03-24